

The Omnipod DASH®
Insulin Management System



Nadia

PODDER™ SINCE 2018

Experience Freedom with Pod Therapy

SIMPLIFY INSULIN DELIVERY. SIMPLIFY LIFE™




Simple and Discreet

Questions?


CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT

WWW.OMNIPOD.COM >

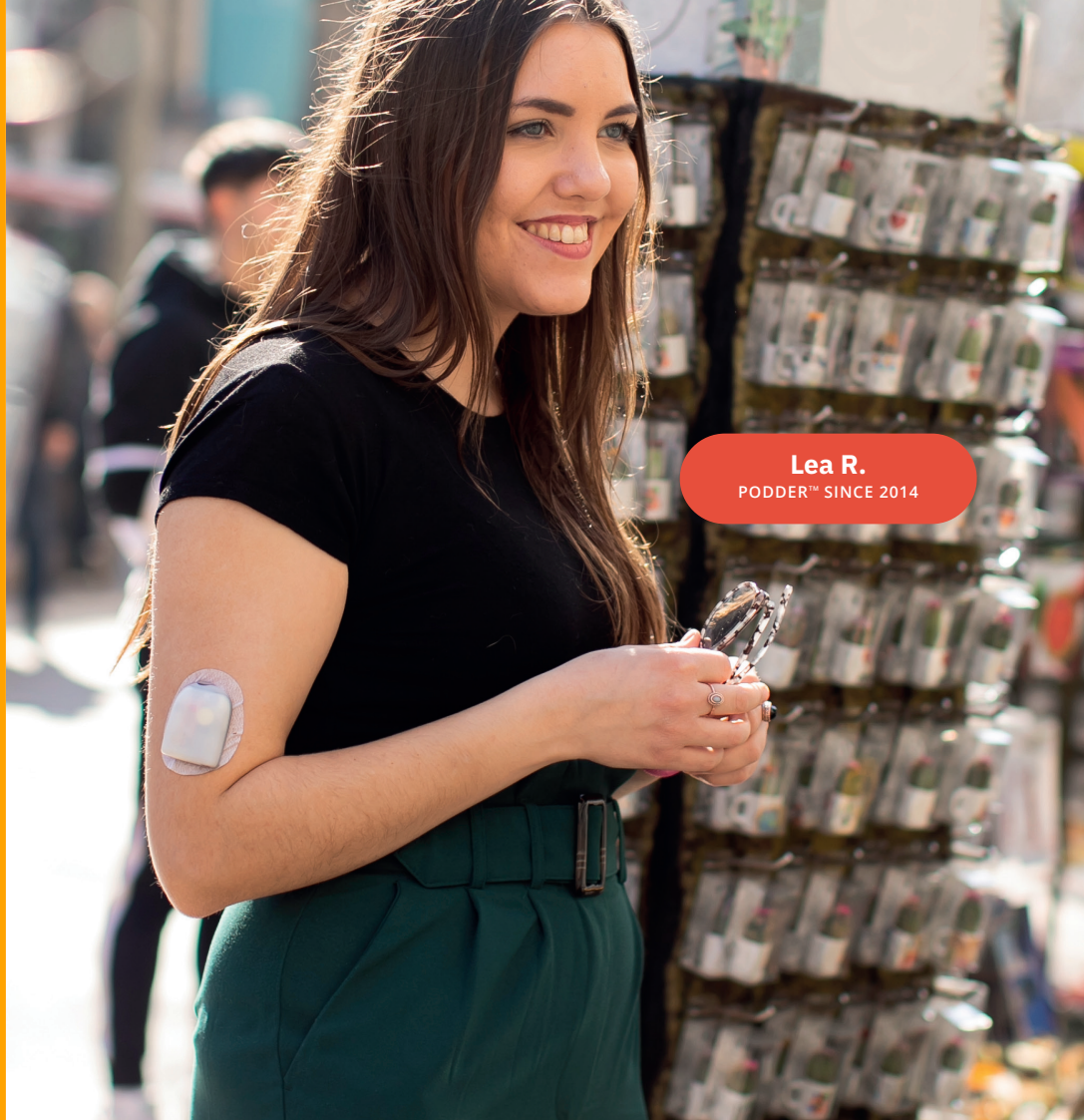

Lea R.

PODDER™ SINCE 2014

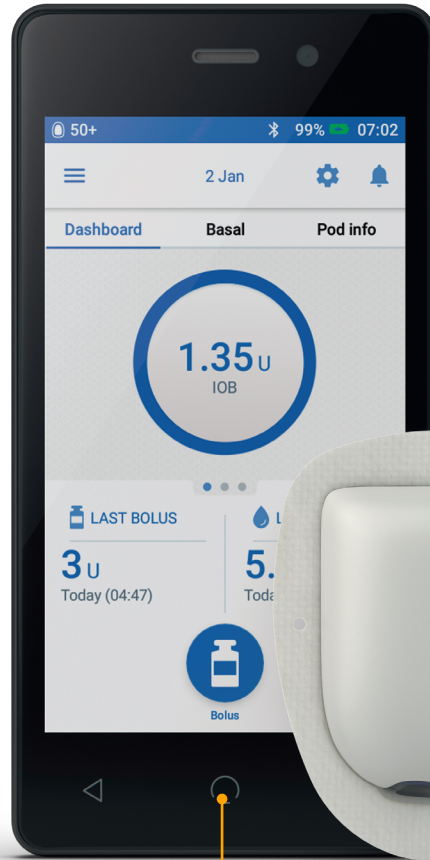

1

A Tubeless, Waterproof* Pod

Free yourself from daily injections, tubing hassles, and wardrobe compromises.



2

A Touch-Screen Personal Diabetes Manager (PDM)

Discreet insulin delivery with a few finger taps.

*The Pod has an IP28 rating for up to 7.6 metres for 60 minutes. The PDM is not waterproof. Screen image is an example, for illustrative purposes only.

So simple to get started

Questions?

**CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT**

WWW.OMNIPOD.COM >

1. Fill the Pod

Fill Pod with up to 200 units of insulin.

2. Apply the Pod

The tubeless Pod can be worn almost anywhere an injection would be administered.

3. Tap 'Start' on the PDM

The small, flexible cannula inserts automatically; you'll never see it and barely feel it.





Lucy L.

PODDER™ SINCE 2009

Fine-tuned insulin delivery with a few finger taps

Questions?

CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT

WWW.OMNIPOD.COM >

Marcus B.
PODDER™ SINCE 2017



A person is sitting on a wicker chair outdoors. A white dog is sitting on the chair next to them. The background is a brick wall. The person is wearing blue jeans and white sneakers. The scene is bright and sunny.

**Design
inspired
by Podders™**

Spent less time on maths, more time enjoying meals

- > Quick and easy bolus dosing with built-in bolus calculator.

Simple, colourful and intuitive touch-screen

- > Easy to use.
- > Discreet.
- > Familiar smartphone-like technology.

Personalised insulin delivery


- > Ability to set specific Presets.
- > Ability to set Zero Basal Rate.
- > Ability to set Fractional Carb Ratios (0.1g/U).
- > Pod site tracker to support skin integrity and insulin absorption.

Up to 3 days of non-stop insulin delivery to avoid 14 injections*



Omnipod:
1

vs.

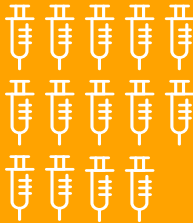

MDI:
14

Consistent, hands-free insertion

- > No need to see or touch an insertion needle
- > The Pod's auto-insertion feature allows for more convenient access to a variety of placement sites.

Discreet insulin delivery

- > The Omnipod DASH® Pod is small, lightweight and discreet.
- > Tubeless – move freely and wear what you want without any concern of a tube getting in the way.
- > No need to interrupt daily activities for manual injections.

Easy treatment modifications

- > Adjust Basal Rates quickly and easily.
- > Deliver insulin in increments of 0.05 units.
- > Stores up to 90 days of insulin data history.

Please note that you should speak to your healthcare provider to assess your suitability for the Omnipod DASH® Insulin Management System.

*Average number of injections required every 3 days (MDI: ~4.5 per day)



Kishon K.

PODDER™ SINCE 2014

Pump benefits, plus Pod comfort and convenience

Questions?

CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT

WWW.OMNIPOD.COM >



Joy S.

PODDER™ SINCE 2014



The Pod is Waterproof* – No need to Disconnect to Shower or Swim

- > Stay connected to your basal insulin delivery – and your favourite activities, without being tied down.

No wardrobe compromises

- > With the Pod you won't have to make clothing choices to accommodate a bulky traditional insulin pump or tubing.

No worrying about Tubes or Site Limitations

- > Having diabetes is difficult enough. Why put up with tubing that can limit your range of motion and pumps that can be a struggle to wear?

*The Pod has an IP28 rating for up to 7.6 metres for 60 minutes. The Personal Diabetes Manager is not waterproof.

Why a Podder™ chose the Omnipod® System?

Questions?

CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT

WWW.OMNIPOD.COM >

Ease of use

- > Sita does not like taking injections.
“With the Pod, I feel a little pinch 1 time every 3 days instead of 5 painful shots every day. It’s so much easier to be compliant with my care.”

“ My focus on my health is so much easier now so I can live, laugh, and enjoy life more. ”

Sita S.

PODDER™ SINCE 2016

This Omnipod® System user testimonial relates to an account of an individual's response to treatment with the Omnipod® System. The account is genuine, typical and documented. However, the individual's response does not provide any indication, guide, warranty or guarantee as to the response other persons may have when using the Omnipod® System. The response other people have could be different. Please speak to your diabetes healthcare professional to assess your suitability for the Omnipod® System.



Make sharing data easy

Questions?

CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT

WWW.OMNIPOD.COM >





Insulet Provided diasend®

- > Gives you and your healthcare providers seamless access to your data with Insulet Provided diasend® Data Management System.
- > Eliminates data upload time during office visits.
- > Enables you and your healthcare providers to collaborate on treatment and make better diabetes management decisions.



Exceptional Customer Service and Support

Questions?

CALL US ANYTIME ON
0800 011 6132[†]
(OUTSIDE THE UK
+44 20 3887 1709)
OR VISIT

WWW.OMNIPOD.COM >



Product Support

- > Available to you 24 hours a day, 7 days a week, even if you are travelling abroad.

FEEL WHAT IT'S LIKE TO
WEAR A POD.
GET A POD EXPERIENCE KIT*
VISIT WWW.OMNIPOD.COM

Screen image is an example, for illustrative purposes only.

Programme offerings

Sample Pod** (Pod Experience Kit)

- > Try out a free, non-functioning sample Pod at home to get the feel of the Omnipod DASH® System before getting started.

Omnipod DASH® System Training

- > Speak with your diabetes healthcare professional to learn more about the training services available to all new Podders™.

Educational and Training Resources

- > How-to videos, resource guides, helpful tips & tricks, and additional educational and training support are available - 24/7 access to educational resources here.

PDM Holiday Loaner Programme***

- > Supporting you wherever you are in the world. PDM Holiday Loaners are offered subject to availability, contact Omnipod® Customer Care Team for more information.

*Pod Experience Kit is not available to order directly by patients in France, Austria and Switzerland. For these countries Pod Experience Kits are only available for healthcare professionals. **The sample Pod is a needle-free, non-functioning Pod that can be worn for up to 3 days. ***Not available in all countries, please contact your local Omnipod® Customer Care Team

Reordering Consumables***

- > Pods can now be ordered online direct from our website. For guidance on how to reorder Pods and other consumable items, please contact Omnipod® Customer Care team.

Pod Disposal Programme

- > Insulet's Pod Disposal Programme provides Podders™ with an alternative way to dispose of used Pods, so you can help reduce landfill and prevent excess waste. Contact Omnipod® Customer Care Team to learn more about how the Insulet Pod Disposal Programme works and to request your first disposal kit.

Insulet Partnering with Glooko® + diasend®

- > With Glooko + diasend®, users have access to all of your diabetes information in one easy-to-use platform. Insulet provided diasend® allows you to review your diabetes patterns on your smartphone or personal computer, understand the cause-and-effect relationships of your activities on your blood glucose and easily share diabetes data with your diabetes healthcare professional.

Technical Specifications

Pod Design

Size

3.9cm wide x 5.2cm long x 1.45cm high

Insulin

Holds up to 200 units of U-100 rapid acting insulin

Average Weight

26g without insulin

Personal Diabetes Manager Features

Alarms

Adjustable volume and vibration alarms

Alerts/Reminders

BG Reminder

Pod Expiration

Low Reservoir

Bolus Reminders

Program Reminders

Other Features

4.0" LCD Touch Screen

Bolus Calculator

Bluetooth® Wireless Technology

Holds up to 90 Days of Data

Bolus Insulin Delivery

Bolus Increments

0.05 units

Correction Factor

1:1 to 1:400 mg/dL in 1/mg/dL increments

0.1 to 16.7 mmol/L in 0.1 mmol/L increments

Insulin to Carb Ratio

1:1 to 1:150 g with 0.1 g increments

Maximum Bolus Size

30 units

Basal Insulin Delivery

Minimum Basal

0.00 units/hour

Maximum Basal

30 units/hour

Basal Increments

0.05 units/hour

Temporary Basal Duration

30 minutes to 12 hours

30 minute increments



Kelly R.

PODDER™ SINCE 2019

Important Safety Information:

The Omnipod DASH® Insulin Management System is intended for subcutaneous delivery of insulin at set and variable rates for the management of diabetes mellitus in persons requiring insulin.

The Omnipod DASH® System has been tested and found to be safe for use with the following U-100 insulin: Novolog®/NovoRapid®, Humalog®, Fiasp®, Admelog® or Apidra®. Refer to the Omnipod DASH® Insulin Management System User Guide for complete safety information including indications, contraindications, warnings, cautions, and instructions.




WWW.OMNIPOD.COM >



† Calls may be monitored and recorded for quality purposes. Calls to 0800 numbers are free from local landlines, but networks may charge for these calls.

©2020 Insulet Corporation. Omnipod, the Omnipod logo, DASH, the DASH logo, Simplify Life and Podder are trademarks or registered trademarks of Insulet Corporation in the USA and other various jurisdictions. All rights reserved. Glooko and diasend are trademarks of Glooko, Inc. and used with permission. The Bluetooth® word marks and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Insulet Corporation is under license. All other trademarks are the property of their respective owners. The use of third party trademarks does not constitute an endorsement or imply a relationship or other affiliation. 1 King St, Hammersmith London W6 9HR. INS-ODS-08-2019-00021-V6

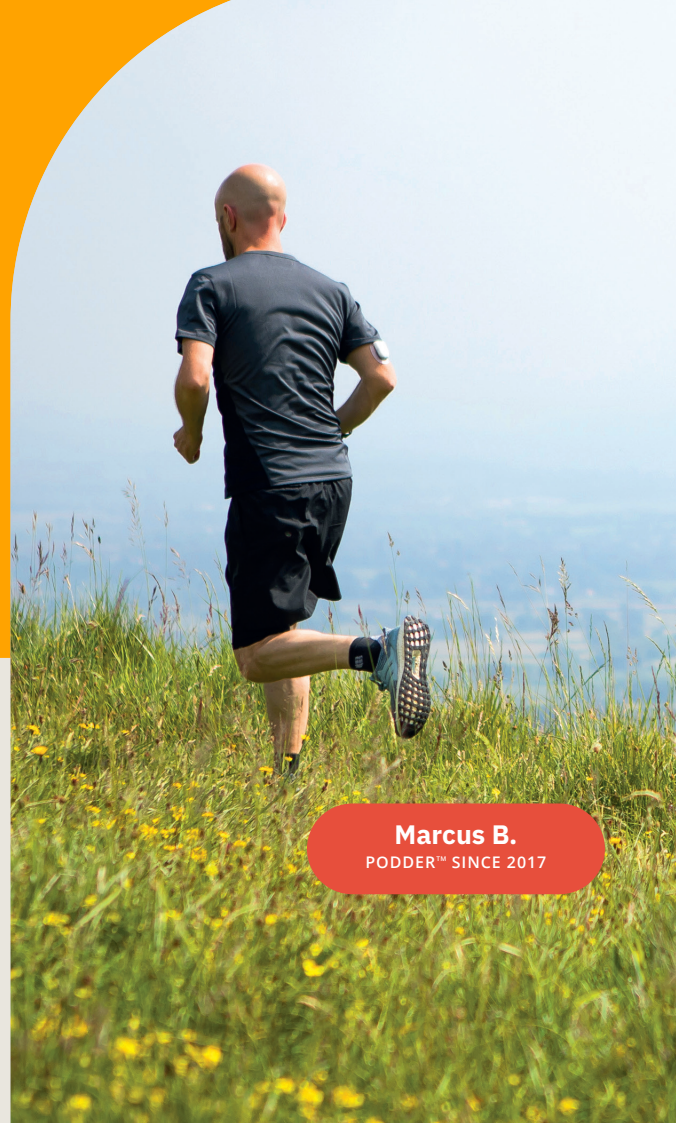

Marcus B.
PODDER™ SINCE 2017